

# IMPRESSION TIPS

Instructions  
on back →

## Common Error #1: Putty too warm.

**Solution: Keep putty in refrigeration for at least 30 minutes before mixing. Then mix thoroughly for 90 seconds.**




**PLEASE NOTE:** Warm putty hardens much quicker so make sure it's cold when you mix it. Mixing it when cold allows you to mix it for a full 90 seconds at a comfortable pace.

## Common Error #2: Bite is too shallow and not centered.

**Solution: Fill tray with putty to the front edge.**

- **Make sure to fill in the tray with putty to the front edge!**
- With the putty in the tray you will need to **press it firmly into your teeth** to make a proper impression.

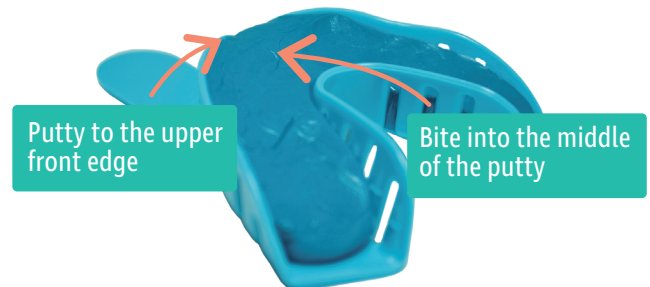

## **Practice Tips**

- Practice putting the tray in your mouth and pressing it up (or down) onto your teeth.
- On your upper teeth, the tray might press into the roof of your mouth, which may be unpleasant.
- This is normal. When you put the putty in the tray it will feel better.
- In the tray, your teeth should not touch the edges.



Need help? Contact us at [support@snapcorrect.com](mailto:support@snapcorrect.com)  
or watch the **"how-to"** video online at [www.snapcorrect.com/goodimpressions](http://www.snapcorrect.com/goodimpressions)

# READ CAREFULLY!!!

Watch the “how-to” video online at [www.snapcorrect.com/goodimpressions](http://www.snapcorrect.com/goodimpressions)

**Ready?** It’s easiest to do this with a friend to help you with the timing.



## Let's Go!

1. Putty must have been in **refrigeration for at least 30 minutes** prior to use.
2. Have the impression tray ready.
3. Put your gloves on.
4. Open both putty containers.
5. Take both putties out of containers.
6. Have your friend start the timer!
7. Using only your **fingertips**, mix for a **full 90 seconds**. The putty should be one color when finished mixing.
8. Roll putty in your palms into a 4-inch log (10 seconds).

9. Carefully press the log into the tray, **making sure the putty goes up to the upper front edge of the tray (15 seconds)**.
10. Center your teeth over the putty, and press the tray **GENTLY** and **DEEPLY** making sure putty covers the gumline.
11. Hold for 3 1/2 more minutes. Then gently remove without rocking the tray.




Remember to repeat these steps for each tray.

## Inspect your Impressions:

- If your gumline is not clearly visible, then you didn't go deep enough. (See page 7 in your Impression Guide.)
- If you see your teeth have hit the plastic bottom, then you have gone a little too deep – go a bit lighter on your 2nd try, but make sure the putty goes over the gumline.
- Don't worry about going too deep. Remember you have 2 chances.

### GOOD IMPRESSION

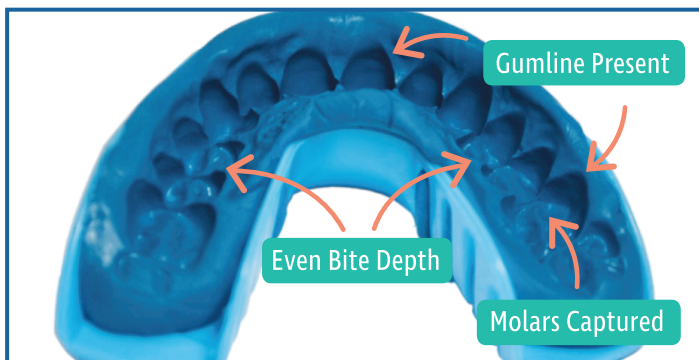

### POOR IMPRESSION

